



信安  
智囊

S-AIRBAG

# S-AIRBAG HIP PROTECTOR S10 USER GUIDE

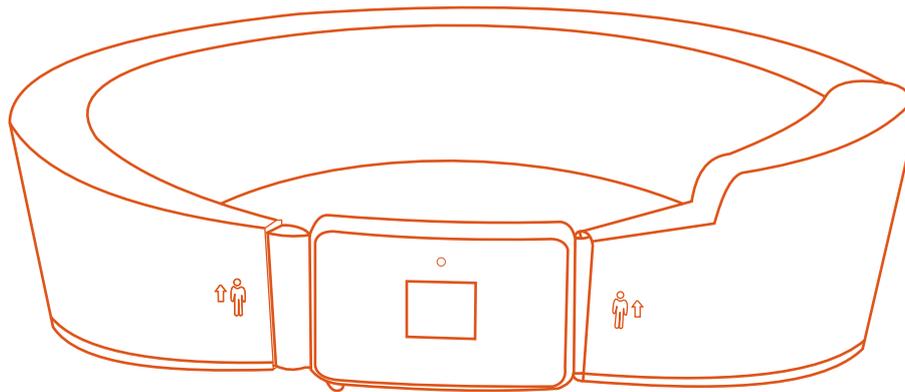

S-AIRBAG, PROACTIVE BODY PROTECTION EXPERT!

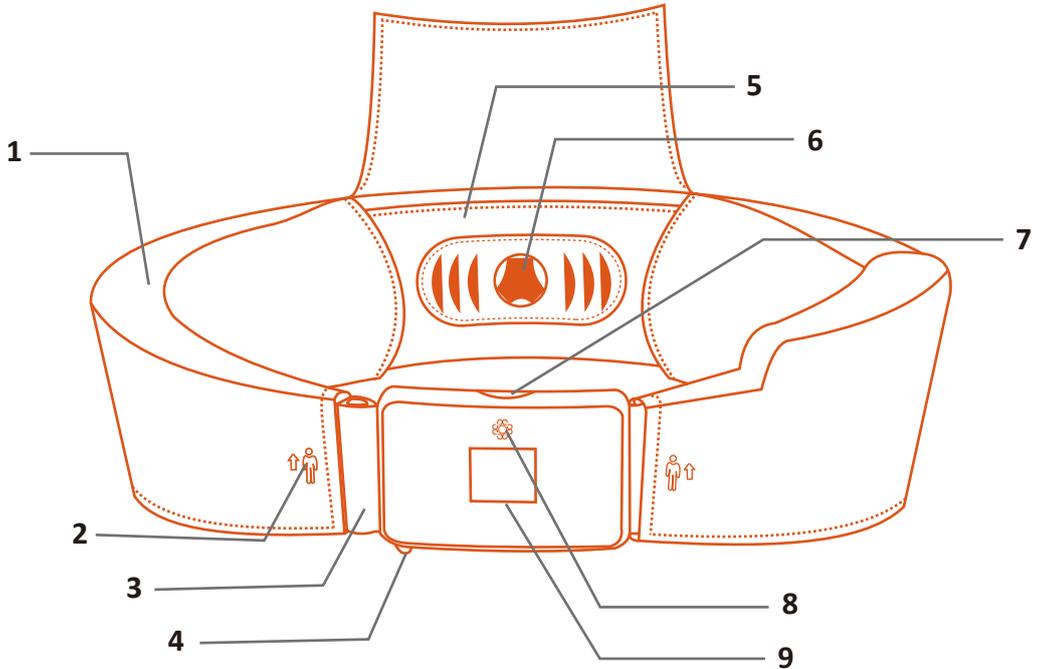

**1. Waterproof Outer Cover**  
**2. Wearing Direction**  
**3. Belt Buckle**

**4. Switch Button**  
**5. Lining Cloth**  
**6. Adjustment Wheel**

**7. Activation Indicator**  
**8. Intelligent Sensor**  
**9. Display Screen**


# HOW TO WEAR

**LOOSE**


Pull out the adjustment wheel and stretch S-AIRBAG Belt to the desired size. Then lock the adjustment wheel by pressing it down again.



Lift

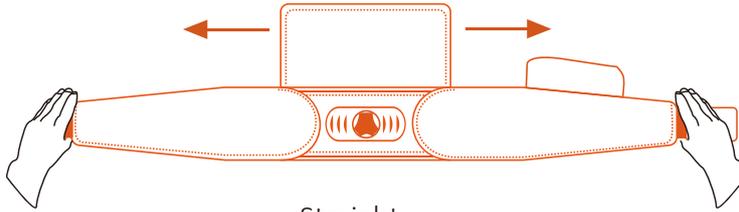

Straighten

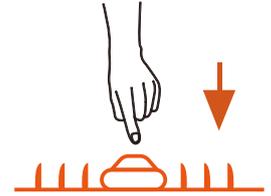


Press


**TIGHT**

Using the adjustment wheel by turning it clockwise until you get a tight fit directly against the waist during use.




Adjustment Wheel

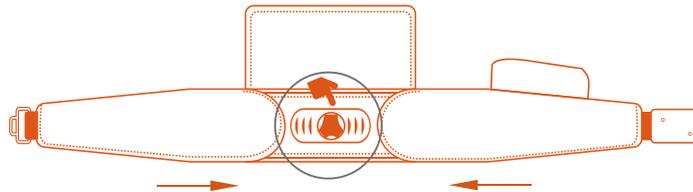

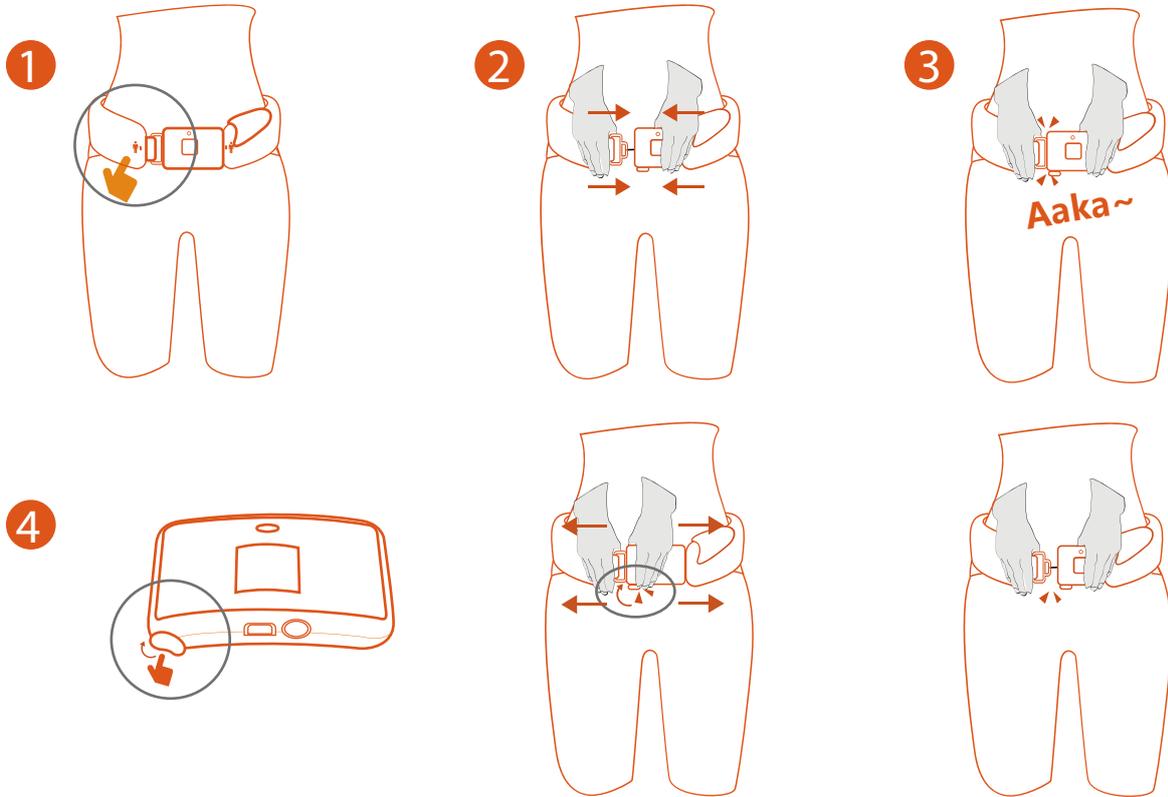

Buckle the belt after you adjust the fit by using the adjustment wheel.  
Unbuckle the belt by turning the switch bottom to the left front.

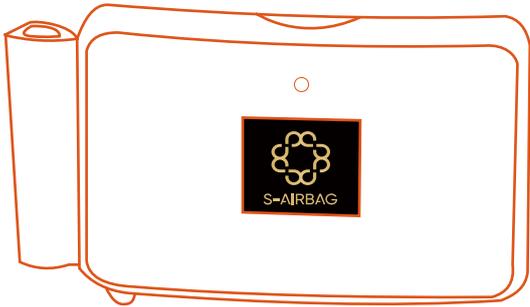

a



b



c



d



e



f





g



h

- When you buckle the belt, Sign (a) will show up immediately. There will be an auto test with an audio signal and the front LED will show the sign(b). It will continue for a short period and change into sign (c) when you wear the belt properly. If you don't wear the belt properly, Sign(b) with the audio signal will last till it is properly worn. Please adjust the fit by using the adjustment wheel to your own size and make sure the wearing direction is correct.
- The LED Display will show the time and battery level when you wear it properly. Please charge the battery when Signal (e) appears.
- Signal(g) means there is some problem with the device, Please contact the customer service for
- After the airbag inflated, the SOS alarm (Signal h)will be displayed, and the associated APP will also receive the corresponding alarm.
- The Screen will be off after a while automatically to save energy. You could wake up the screen by putting your hand close to the screen.

## CHARGING THE BATTERY

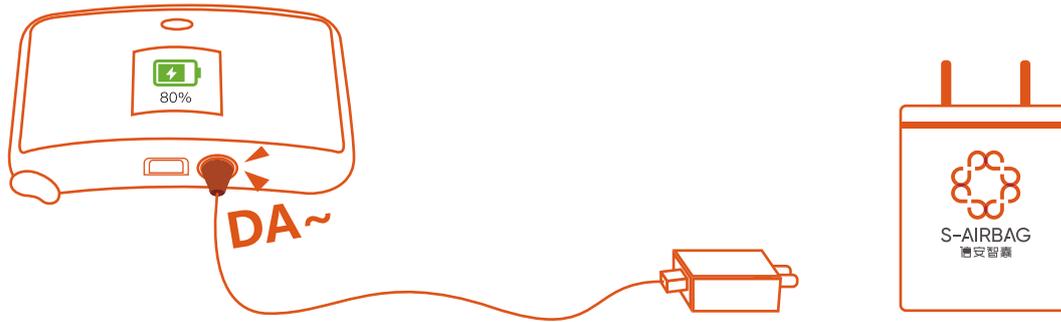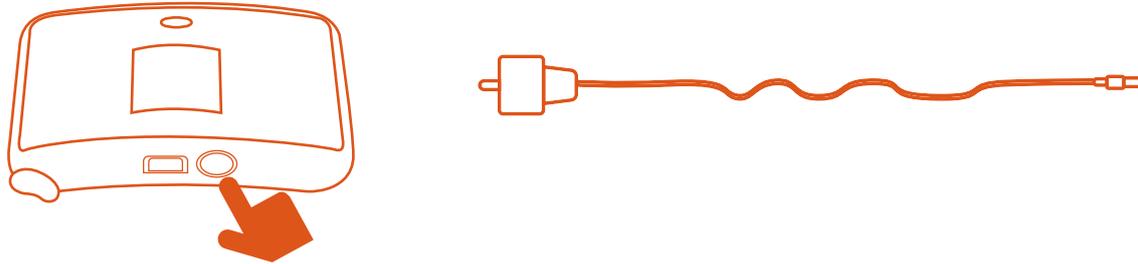

The device needs to be charged when the battery is low. It can be charged through the magnetic suction port. You could wake up the screen and check the battery level. The LED light (7) will indicate the battery level as follow:

Solid green light means the device has enough power.



Solid red light means low power.

Flashing red light means the device needs charging immediately.

# Internet Function

Scan the above QR code or download the S-AIRBAG APP in the APP Store directly. You could sign up with your information and follow the below steps to register your device.  
[My Account]-[Associated Device]-[Scan Code] to register your device.

**Download software  
qr code**

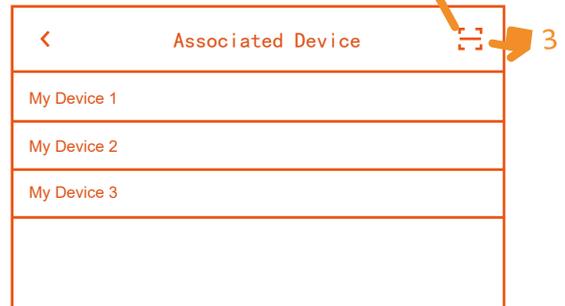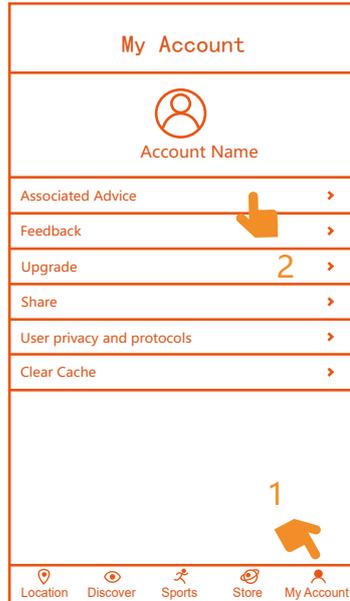


After the airbag inflated and protected the user, the associated APP will receive the alert information with buzzing beeps and show the specific location.




The device display screen's status



The content shows on the APP

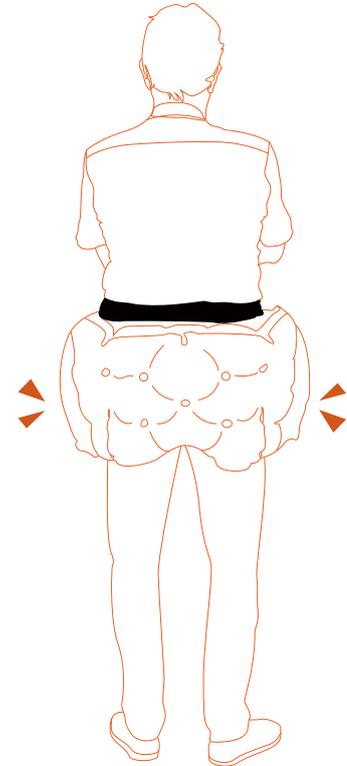

Deploying status of the airbag

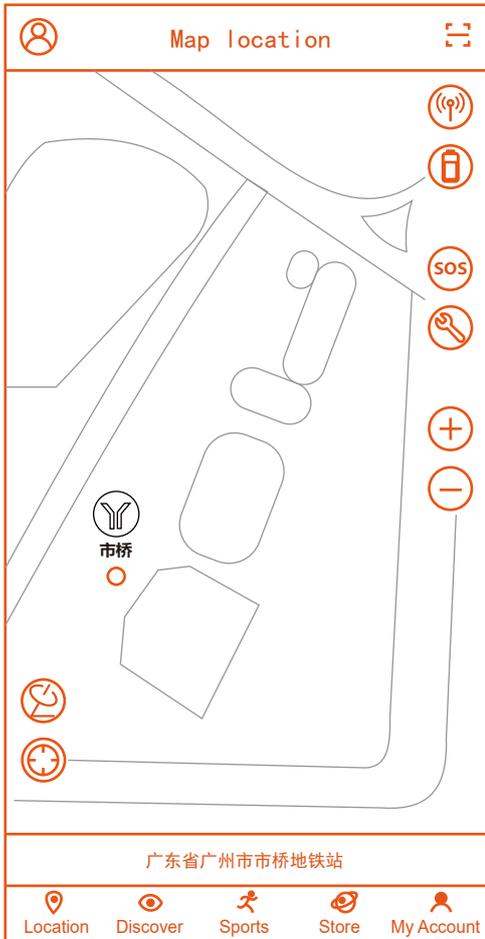

## Satellite Positioning

Real-time satellite positioning



## Walking Track

Positioning to the current device location



## Device Signal

Display the device signal condition



## Device Power

Display the device battery status




## SOS Alarm

After the device triggers an alarm, the associated APP will receive the device alarm, including alarm location. Associated family members can contact immediately or take rescue measures (this feature requires permission that this program keeps running in the backend of the phone)



## Malfunction Alarm

When there is a malfunction of the device, the mark will be displayed in red, and a fault message will pop up on the display screen. When the screen is off, you can press the malfunction button to re-show the fault message again.

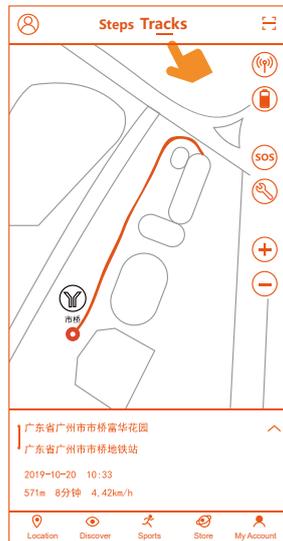

## Steps & Tracks

1. Click [Sports] to view the step record of the day;
2. Click [week / month] to switch the duration, you can view 7 days' track record
3. Click [Sports]-[Track] to view the track record.

## SOS Alarm

When the wearer falls down, the airbag will be triggered to protect the user. The SOS alarm with buzzer beep will appear on the screen and the associated APP will pop up an alarm notice page simultaneously. In the meantime, the fall location and the optimal navigation route will be shown to the caregiver for immediate rescue.

# MAINTENANCE

- Check your S-AIRBAG HIP PROTECTOR is in good condition, i.e. no rips or tears, before each use. If some damage occurs, please contact the authorized dealer or our customer service team for further information.
- Once the airbag is deployed, the gas cylinder must be replaced with a new one of the same capacity. Contact the authorized dealer or our customer service team for support.
- Do not remove or disassemble any part of the internal structure of the device by yourself.
- Please use our original power adapter only and keep the device away from fire or other heat sources and sharp objects.
- Do not wear the device when you are bathing or sleeping.
- This device contains battery inside, please keep it separately with other items. Please fully charge the battery when it is not used for a long time.
- This device contains delicate parts and should be handled carefully. Keep out of reach of children.
- Do not sew/iron or stick anything on the device as it could pierce the airbag and prevent them from working properly.

## **AFTER INFLATION**

The gas cylinder inside is to trigger the airbag in case of fall and for single use only. After the device's airbag is deployed, the gas cylinder must be replaced and the airbag need to be folded after each fall.

You could replace the consumables by yourself by referring to the TUTORIAL VIDEO on our website or contact your authorized dealer for support. Please do make sure you replace the gas cylinder and fold the airbag correctly.

### TUTORIAL VIDEO

For detailed tutorial video, please visit our website: [www.s-airbag.com](http://www.s-airbag.com)

\*Reuse the belt: follow this video to fold the airbag and replace the gas cylinder

# DISCLAIMER

Dear user, please learn more about the products features before installation and usage, so that our products can work properly and give you protection.

1. when you wear properly, it will have the work signal indication. If there is fault warning during normal using, please contact our after-sales service and follow the instructions to eliminate the fault. if it can not be solved, please send the device back to the factory to repair.

2. Please note that no body protector can prevent death or serious injury in certain accidents. This device is not a full-body protection device, it absorbs shocks and offers the collar of the hip and pelvis more support, thus reducing the risk of hip fractures.

3. The triggering protection of the airbag is based on the back or side fall of the wearer in a normal standing posture or walking state. But for other situations, such as holding the handrail to fall slowly when falling down or falling in a sitting or squatting position, the device may not deploy the airbag to protect because of its slow falling speed. Please be noted that if the front direction fall happens, the device will not deploy the airbag.

4. For severe osteoporosis patients, fractures may occur even with a slight collision. Therefore, it is recommended to seek medical treatment to avoid the risk of injury.

5. Please take protective measures during non-product wearing periods (including but not limited to anti-slip in the bathroom and protection of get up in night).

6. All S-AIRBAG PROTECTIVE BELTS are rigorously tested before being sold. Every airbag belt is inflated to reduce the fall pressure as it would be when inflated normally, however we cannot guarantee the airbag belt will inflate every time or be held responsible should it not.

S-AIRBAG cannot be held responsible should the protective belt not inflated.

7. when you are taking in a vehicle or other transportation, the device cannot provide you effective protection. It is recommended not to wear any garment or backpack over the device.

8. When you are engaged in intense sports such as running and dancing, please do not use this product because the built-in sensor of the product may detect that you are about to fall and deploy the airbag to protect. In addition, it is not recommended for people with implantable electronic medical devices such as cardiac pacemakers.

S-AIRBAG will be held in no way responsible for any bodily injury or death, and/or property damage that may occur whilst using any S-AIRBAG products.

For more information, please visit [www.s-airbag.com](http://www.s-airbag.com) or send email to [service@s-airbag.com](mailto:service@s-airbag.com)